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REF

69219 Anti-C^w incomplete, polyclonal, human 5 ml

730-13-9702 Version 002 / 20. Januar 2022

BAG
DIAGNOSTICS

BAG Diagnostics GmbH
Amtsgerichtsstr. 1-5
35423 Lich/Germany
A BAG Group company

T. +49 (0) 6404/925-100
F. +49 (0) 6404/925-460
M. info@bag-diagnostics.com
W. www.bag-diagnostics.com

Ordering:
T. +49 (0) 6404 / 925 - 450
F. +49 (0) 6404 / 925 - 460
M. order@bag-diagnostics.com

Customer Service:
T. +49 (0) 6404 / 925 - 125
F. +49 (0) 6404 / 925 - 421
M. service@bag-diagnostics.com

Anti-C^w incomplete, polyclonal, human

For Slide- and Tube Method

FOR IN VITRO DIAGNOSTIC USE ONLY

English

INTENDED USE

Polyclonal Coombs-reactive Anti-C^w reagent is produced from human plasma that contains antibodies of IgG-type which reacts specific with the corresponding blood group antigen. The reagent is used for In-Vitro-Diagnostic, to determine qualitative, whether human red blood cells possess or lack the corresponding blood group antigen C^w. The reagent is intended to be used by qualified and technical personnel only.

PRINCIPLE OF PROCEDURE

The test methods used with this reagent are based on the principle of agglutination. Normal human erythrocytes, possessing the appropriate antigen, will agglutinate in the presence of the specific antibody directed toward the antigen.

REAGENTS

The reagent is available in following formulation:

Anti-C^w incomplete, polyclonal, human

The reagent contains <0.1% (w/v) sodium azide as preservative. Beside the parts active antibody and human serum, the reagent contains sodium chloride, macromolecules and bovine albumin, which has been tested and certified by the US Veterinary service inspectors.

WARNING

The reagent is prepared from human plasma. The raw materials for this reagent have been tested for HBsAg, HIV1/2- and HCV-antibodies and found to be negative. Nevertheless, as biological product it should be looked upon as potentially infectious because of never complete exclusion of danger through excipients of disease. The reagent contains sodium azide that may be toxic and may react with lead or copper to form highly explosive salts.

On disposal, flush with large quantities of water. For the reasons mentioned above, the reagent should be handled with proper care.

STORAGE REQUIREMENT

Store opened and unopened products at +2 to +8°C.

May be at room temperature while in use.

In principle, store and use the reagent to indicated expiry date only.

REMARKS

1. With each testing positive and negative controls should be performed.
2. Inappropriate storage impairs efficacy of the reagent.
3. Weak turbidity of the reagent does not affect its reactivity. Bacteria and chemical contamination of the product should be avoided. If a visible change is detected, it may indicate a microbiological contamination, the reagent should no longer be used.
4. Strength of positive reactions also depends on age of used blood.
5. Centrifugation outside the specified speed range may lead to false results.
6. The test method identified below is for manual testing only. When using automated or semi-automated instruments, follow the procedures that are contained in the operator's manual provided by the device manufacturer. Laboratories must follow approved validation procedures.
7. For usage of this reagent all effective national laws, directives and guidelines have to be observed in its current version, in Germany especially the „Richtlinien zur Gewinnung von Blut und Blutbestandteilen und zur Anwendung von Blutprodukten (Hämotherapie)“¹.

SAMPLE PREPARATION

1. Blood sample should be collected by approved medical procedure.
2. Blood sample to be tested should be used as soon as possible after blood collection to reduce the risk of false-positive and false-negative results due to improper storage or contamination of the sample.
If a delay in testing occurs, sample should be stored at +2 to +8°C.
Blood drawn into EDTA should be tested within 7 days and sample treated with sodium citrate within 14 days after collection.
Blood bag / Donor Blood can be tested until the expiry date.

REAGENT PREPARATION

There is no preparation of the reagent required. Take and use reagent directly from the vials.

PROCEDURE

Not provided material, additionally needed

Slide Method:

1. glass slide
2. pasteur pipette
3. mixing stick
4. timer
5. incubator
6. cover hood
7. AB human serum
8. isotonic saline (0,85 - 0,9% sodium chloride)

Tube Centrifugation Method:

1. tubes (10 x 75 mm or 12 x 75 mm)
2. microliter pipette
3. timer
4. incubator
5. centrifuge
6. isotonic saline (0,85 - 0,9% sodium chloride)

Test procedure

Slide Method

1. Prepare a 10% erythrocyte suspension in AB human serum (red blood cells may be washed 1-3 times with isotonic saline) or use whole blood.
2. Place one drop (approximately 50 µL) of the reagent on a marked glass slide.
3. Using a Pasteur pipette add one drop 10% erythrocyte suspension or whole blood (approximately 50 µL) to the drop of reagent on the glass slide.
4. Mix the erythrocytes with reagent well with a stick and spread to a circle with a diameter of approximately 2 cm.
5. Incubate glass slide covered at +37 °C in an incubator for 15 min.
6. By slightly rotating the slide, check for agglutination
7. Document the result.

Tube Centrifugation Method

1. Prepare 2% to 5% suspensions of red blood cells in isotonic saline (red blood cells may be washed 1-3 times with isotonic saline).
2. At first put 100 µL of the reagent in a marked tube, subsequently add 100 µL of appropriate cell suspension.
Alternative one drop = approximately 50 µL cell suspension can be added to one drop = approximately 50 µL reagent.
3. Mix Erythrocytes- / Reagent-mixture well by slightly shaking.
4. Incubate tube in an incubator at +37 °C for 30 min.
5. Centrifuge tube for 1 min at 2.000 rpm (approximately 800-1.000 x g).
6. Gently shake the red cells from the bottom of the tube and check macroscopically for agglutination within 3 minutes.
7. Document the result.

INTERPRETATION OF RESULTS

"Slightly rotating" at Slide Method

"Slightly shaking" at Tube Centrifugation Method:

Positive results (+): visible agglutination of erythrocytes is a positive result and indicates the presence of the corresponding antigen.

Negative results (-): No visible agglutination of erythrocytes is a negative result and indicates the absence of the corresponding antigen.

LIMITATIONS OF THE PROCEDURE

1. Inaccuracy at compliance with instructions written under section "Procedures" and "Interpretation of results" may lead to incorrect results.
2. No valid conclusion concerning the test result can be reached, if controls with uncertain or false results occur.
3. Enzyme treated erythrocytes or addition of bovine albumin and / or other solutions containing protein may cause unspecific reactions.
4. Hemolyzed, turbid, contaminated or clotted samples should not be used.
5. With the slide method, unspecific reactions might appear due to drying of the reaction-formation.
6. Due to variability of antigen expression, reactivity of this reagent against certain phenotypes may give weaker reactivity compared to control cells.
7. No one specific antiserum or technique can be guaranteed to detect all rare, weak or variant Antigens.²
8. Red blood cells coated with alloantibodies or autoantibodies of the same or similar specificity as that reagent used for the test (i.e., cells that are positive in the direct antiglobulin-test (DAT)) are unsuitable.
9. Red blood cells with a positive direct Coombs-test may cause false-positive reactions. The cells react positively even without a test serum.

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 REF	Product Code	 LOT	Lot
	Store from - to		Expiration Date
 IVD	In-Vitro Diagnostic		EU CE symbol
	Manufacturer as to 98/79/EG		Observe instruction for use
 UDI	Unique Device Identification		Distributor

REF

69219 Anti-C^w incomplete, polyclonal, human 5 ml

730-13-9702 Version 002 / 20. January 2022

CE

BAG
DIAGNOSTICS

BAG Diagnostics GmbH
Amtsgerichtsstr. 1-5
35423 Lich/Germany
A BAG Group company

T. +49 (0) 6404/925-100
F. +49 (0) 6404/925-460
M. info@bag-diagnostics.com
W. www.bag-diagnostics.com

Ordering:
T. +49 (0) 6404 / 925 - 450
F. +49 (0) 6404 / 925 - 460
M. order@bag-diagnostics.com

Customer Service:
T. +49 (0) 6404 / 925 - 125
F. +49 (0) 6404 / 925 - 421
M. service@bag-diagnostics.com

ANTITOXIN GmbH Industriestraße 88 69245 Bammental Germany