

# TERASAKI Park MEDIUM

Cell culture medium for storage of cells (e.g. lymphocytes)  
 Electronic instructions for use see [www.bag-diagnostics.com](http://www.bag-diagnostics.com)

**REF** 7028

## 1. DESCRIPTION OF PRODUCT

The intended use of **TERASAKI Park MEDIUM** is the storage of cells, e.g. lymphocytes. The solution is ready to use. It is suitable to maintain physiological conditions for cells, e.g. human lymphocytes. In **TERASAKI Park MEDIUM**, isolated cells (e.g. lymphocytes) may be kept up to 4 days without noticeable loss of viability.

## 2. APPLICATION (e.g. for lymphocytes)

Isolate lymphocytes as usual. After the last washing step, the sediment is topped up with 5-10 ml of **TERASAKI Park MEDIUM** and stored refrigerated at 2...8°C.

In order to receive the required amount of cells for e.g. HLA-typing proceed as follows:

1. Centrifuge the cell suspension for 10 minutes at 110 x g.
2. Discard the supernatant.
3. Adjust the sediment with **TERASAKI Park MEDIUM** to the required amount of cells (e.g. 2000-3000 cells/µl).

## 3. WARNINGS AND PRECAUTIONS

Do not use **TERASAKI Park MEDIUM** in case of changing the colour from orange to yellow.

Do not use **TERASAKI Park MEDIUM** beyond the indicated expiration date on the label.





A declaration on Material Safety Data Sheets (MSDS) are available to download at [www.bag-diagnostics.com](http://www.bag-diagnostics.com).

**4. SHELF LIFE:** until the expiration date indicated on the label,  
 after opening to be used within 1-2 weeks

**5. STORAGE:** 2...8°C

**6. PACKAGE:** 100 ml

## 7. EXPLANATION OF SYMBOLS USED ON LABELLING

	Storage temperature / Temperature limitation
	Use by
	Consult Instructions for use
	Manufacturer
<b>CELLCULTURE MEDIUM</b>	Cellculture medium
<b>CONT FCS 0.5%</b>	Contains 0.5% foetal calf serum
<b>LOT</b>	Batch code
<b>REF</b>	Catalogue number

Instructions for use in other languages see <http://www.bag-diagnostics.com>  
 or phone +49 (0)6404-925-125

Version: 4 / 2022 / Issue: 2022-09

