


Special Requirements for RT-PCR Cycler Systems

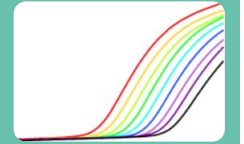

RT-System	Custom dye calibration / color compensation	Special notes
BioRad CFX96	No	
Roche LC 480 I + II	Yes - color compensation	
Roche LC 2.0	Yes - color compensation	ROX 120 nM in PCR mix required
Roche LightCycler 96	No	
QuantStudio 1, 3 & 5, 6 & 7, 12K	Yes – dye calibration	
StepOne & StepOne plus	Yes - dye calibration	
ABI 7500 & 7500 fast	Yes - dye calibration	
ViiA 7	Yes - dye calibration	
Agilent AriaDx & Mx	No	
Agilent Mx3000 & Mx3005P	No	
MIC	No	
Qiagen Rotor-Gene Q	No	USB cable for Q Rex software required

If your RT-System needs a custom dye calibration or a color compensation please contact our Service Team:

Phone.: +49 6404 – 925125

Email: complaint@bag-diagnostics.com